DELTA 4

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THE EVOLUTION FROM HOSPITAL BED TO WORKING TOOL

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DELTA4 is the new generation of beds developed according to the needs of healthcare professionals, with an idea in mind: if the bed allows the operators to work at their best, the patient's well-being follows naturally.

With **DELTA4** the bed becomes an active platform that simplifies and maximises the performances of healthcare professionals.

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AN EVOLUTIONARY LEAP THAT STARTING FROM PROFESSIONALS

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DESIGNED FROM THE MOST DEMANDING SITUATIONS, A SOLUTION FOR EACH DEPARTMENT

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Working alongside people who work in critical situations, we realized how crucial the bed is.

We asked ourselves, how much can it improve care if the bed increases the operator's skill?

Designing a bed around a caregiver is no longer a limitation, but a powerful and versatile tool.

From this fieldwork we have made innovations that make the **DELTA4** bed the perfect ally in all care conditions.

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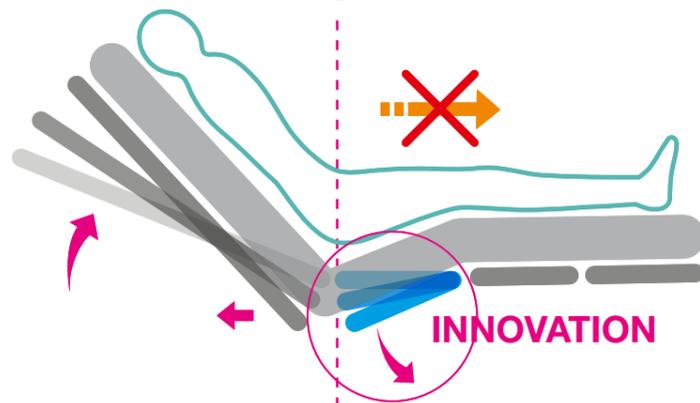
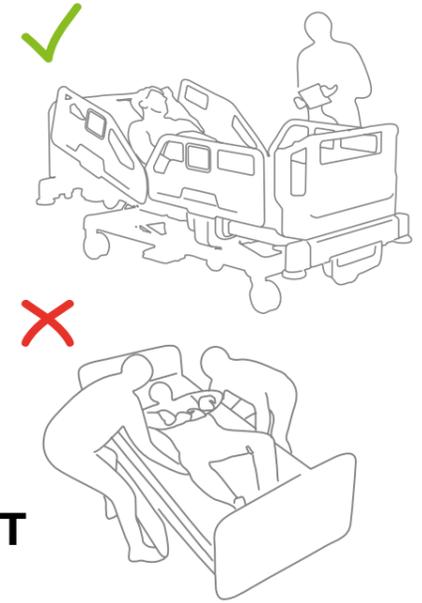
NO STRESS NO HASSLE, MORE TIME FOR WHAT MATTERS MOST

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How much time and effort can you save if the patient doesn't need continual attention in bed?

Those who work in this field know this well: the patient sliding towards the foot of the bed is an ongoing problem. The continuous slipping and repositioning cause discomfort to the patient and produce a friction that in the long run can lead to bedsores. Furthermore, repositioning involves not one, but two people at a time, in strenuous and dangerous operations for both caregiver and patients. We solved this critical point with the INCENTRO SYSTEM: **the combined back and pelvic translation nullifies horizontal forces, prevents slipping and keeps the body at a 30° angle, favouring spontaneous breathing.** A patient who does not slip reduces fatigue and the risk of injury, and frees the staff to focus on more useful activities for the patients.

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“MY PATIENTS ARE SAFE EVEN WHEN UNATTENDED”

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Safe exit position button brings the bed to the minimum height to help the patient stand up. The patient can autonomously exit from the bed or be attended by the nursing staff.

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The patient can safely leave the bed during the night without the help of the nursing staff, thanks to the night light.

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Overnight I can immediately verify if the bed height guarantees a safe and autonomous bed exit to the patient. The orange light indicates the unsafe mattress platform height, the green light indicates the safe one.

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High containment safety sides (42 cm) with dampened descent.

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“BED EXIT DETECTION: MY EYES EVEN WHEN I’M NOT THERE”

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The sensor system monitors the patient’s presence and their possible exit by alerting the nursing staff with an acoustic and luminous alert. On request, the warning can be sent remotely. The desired parameters can be set up through the touch screen on the side rail. (System available with active pressure sensors).

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FALLS FROM BED REPRESENT MORE THAN 60% OF TOTAL FALLS

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PATIENT WEIGHT UNDER CONTROL AT ALL TIMES

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The exclusive weighing system was designed to guarantee a high level of precision independently from the mattress platform’s position.

Thanks to the 4 sensors located at the corners of the bed, the load is better distributed and the reading level more accurate.

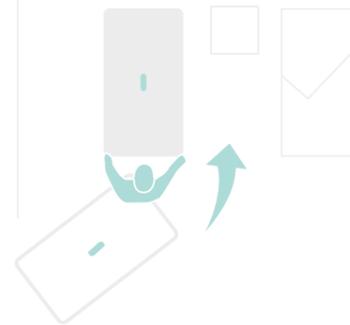
The weighing data can be recorded manually whenever necessary or automatically every 24 hours.

The weighing system allows to set up different types of alerts: exceeded threshold weight alert or “bed exit” alert.

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“ INCREASED MANEUVERABILITY: I CAN MOVE THE PATIENT SAFELY AND EFFORTLESSLY ”

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With the user-friendly Traction Control Switch you can control both the forward/backward movement and walking speed.

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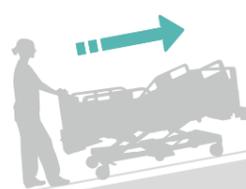
The system is equipped with an anti-crushing sensor that ensures the operator's safety during backwards maneuvers.

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
Thanks to the support, the ascent and descent brake assist of the **DRIVESYSTEM** moving system, I can easily face ramps with up to a 10% incline.

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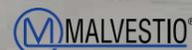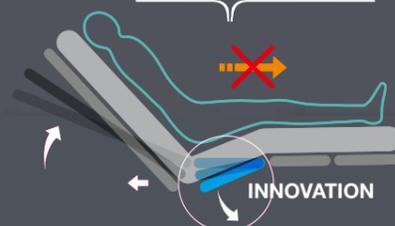
IPX6 CERTIFIED -
HIGH LIQUID PROTECTION

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“**VERSATILE AND SAFE:
EVEN THE MATERIALS HELP ME**”

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The mattress platform can be easily removed; all sections are designed for easy cleaning.

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**SMOOTH AND ROUND SURFACES
WITHOUT SHARP EDGES**

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**X-RAY EXAMINATION
WITH NO NEED TO MOVE THE PATIENT**

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Wide area of radiotransparency. The HPL radiotransparent backrest has an X-Ray cassette holder for chest x-ray examinations, without moving the patient and at any angle of the backrest.

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
The X-Ray tray can also be pulled out with backrest in horizontal position, avoiding to move the patient.

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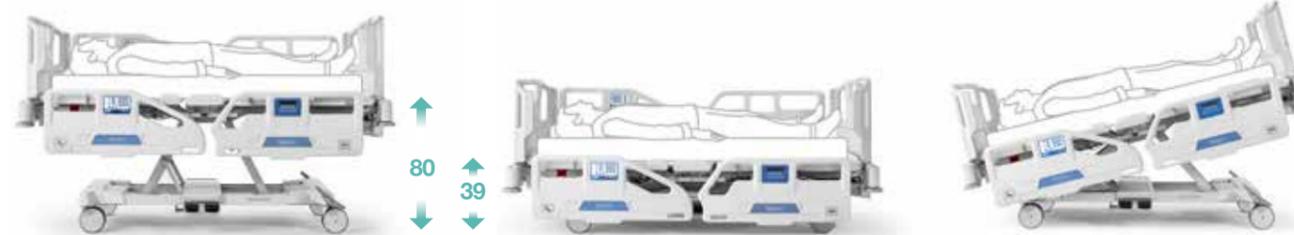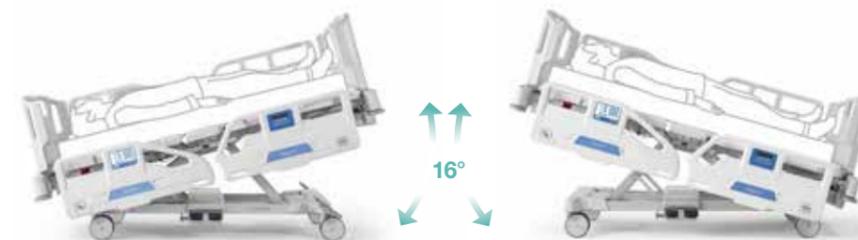
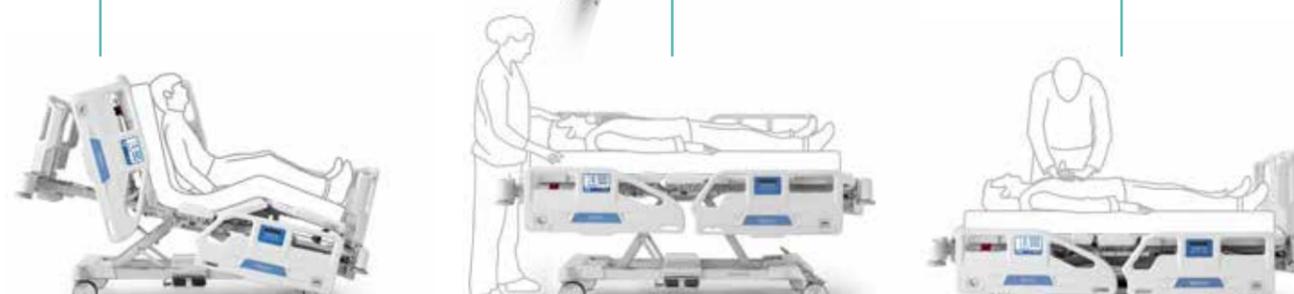
The antishock position is reached quickly, with the simultaneous lowering of the backrest, zeroing of the mattress platform sections and Trendelenburg positioning.

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The chair position helps the patient preventing respiratory complications, it induces to a more physiologic position and has a psychologically positive influence.

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The examination position height adjustment facilitates medical staff in the examination, medication and monitoring operations, by standing up straight.

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The CPR adjustment allows to bring the mattress platform immediately down to the horizontal.

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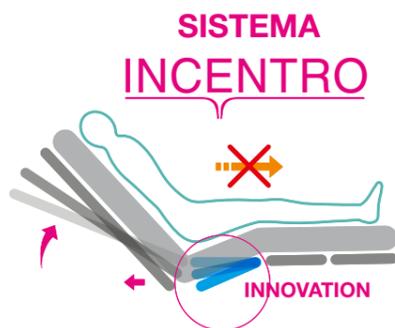
FUNCTIONAL FEATURES

SPALLA VINCOLATA
HEAD-END CONNECTED TO THE INTERMEDIATE FRAME

SPALLA SVINCOLATA
HEAD-END CONNECTED TO THE BASE FRAME

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INCENTRO SYSTEM
The combined back and pelvic movement nullifies horizontal forces, prevents slipping and keeps the body at a 30° angle, favouring spontaneous breathing.

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BED LENGTHENER
The bed lengthener can be controlled by just one person.

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4 SECTOR SAFETY SIDES WITH ELECTRIC CONTROLS
The system of 4 sector safety sides with integrated controls allows full autonomy of movement to the patient.

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SAFE EXIT
The safe exit control moves the mattress platform to the minimum height and lifts the backrest up to facilitate the patient to get up from the bed.

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CPR
The CPR control on both sides allows the bed platform to be brought to the horizontal position immediately; accessible even with the side rail lowered.

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REMOVABLE HEADBOARD AND FOOTBOARD
The bed ends can be easily removed for emergency operations.

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BRAKING SOUND ALARM + ORANGE / GREEN LIGHT

The orange light indicates the unsafe mattress platform height, the green light indicates the safe one. Sound alarm for wheels not broken.

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ELECTRIC FOOT PEDALS

Electric foot pedals on both sides to adjust the mattress platform height.

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BLANKET AND "SUPERVISOR" HOLDER

Extractable tray for blankets and supervisor.

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ENGAGEABLE 5TH WHEEL

Centrally located fifth wheel for U-turns and effortless manoeuvres.

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X-RAY TRAY

The X-ray tray together with HPL backrest allows extremely easy x-ray operations to be carried out

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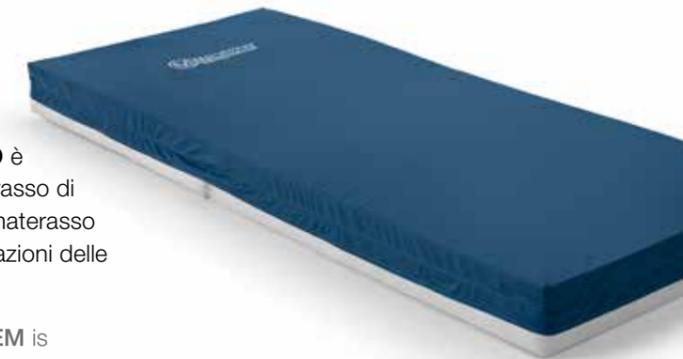
THE MATTRESS ENHANCES THE INCENTRO SYSTEM'S FEATURES

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The INCENTRO SYSTEM is suitable for any type of hospital mattress. By using a dedicated mattress, specific functions are enhanced

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CUSTOMIZATION AND COLOURS FOR EVERY CONTEXT

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DELTA4 bed is available in different colours and can be personalized with the Health Care facility logo.

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Each **DELTA4** bed can be fitted with a wide selection of accessories available to get the perfect configuration for your operating environment.


Trazione cervicale - Cervical traction	Pulsantiera torcia - Hand set with light
Trazione zuppinger - Zuppinger traction	Pedaliere elettriche - Electric pedal controls
Trazione omerale - Humerus traction	Tastiera satellite - Satellite push-button
Trazione pelvica - Pelvic traction	Portabombola girevole - Turning cylinder holder
SISTEMA INCENTRO - INCENTRO SYSTEM	Pianetto porta monitor ad innesto - Tip-over monitor tray / writing desk
Kit ruote Ø125 mm antistatico - Kit of Ø125 mm antistatic wheels	Materasso statico 2 sezioni dedicato - Dedicated 2-section static mattress
Kit ruote Ø150 mm antistatico - Kit of Ø150 mm antistatic wheels	Allungamaterasso statico dedicato - Dedicated static mattress extension
Quinta ruota inseribile - Engageable fifth wheel	Aste sollevamalato - Patient-lifting pole
Quinta ruota elettromotrice DRIVESYSTEM DRIVESYSTEM motorized fifth wheel	Asta sollevamalato con maniglia regolabile mediante sistema avvolgibile a pulsante - Patient lifting pole with adjustable handle
Allungaletto - Bed lengthener	Aste flebo - I.V. pole
Vassoio portacoperte/supervisor - Blanket and "supervisor" holder	Porta sacca urina - Urine sack hook
Vassoio porta lastre - X-RAY tray	Supporto per pappagallo - Urine bottle holder
Sponde a compasso - Folding side-rails	Portapadella - Bedpan holder
Sponde a 4 settori - 4 section side-rails	Portagrafica - Chart holder
Sponde a 4 settori comandi elettrici - 4 section side-rails with electric controls	Bilancia elettronica - Electronic weighing scale
Sponde a 4 settori comandi elettrici con chiamata infermiera 4 section side-rails with electric controls and nurse call	Schienale radiotrasparente in laminato HPL radiotransparent backrest
Allarme freno + Luci giallo/verde Braking sound alarm + Yellow/green lights	

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Malvestio professionals fully train the staff in taking full advantage of the features of this new work tool. The support of specialized personnel covers all areas of use of the bed in its various phases of use: from on-call assistance for any need to scheduled and preventive maintenance to ensure that your **DELTA4** bed is always in full working order.

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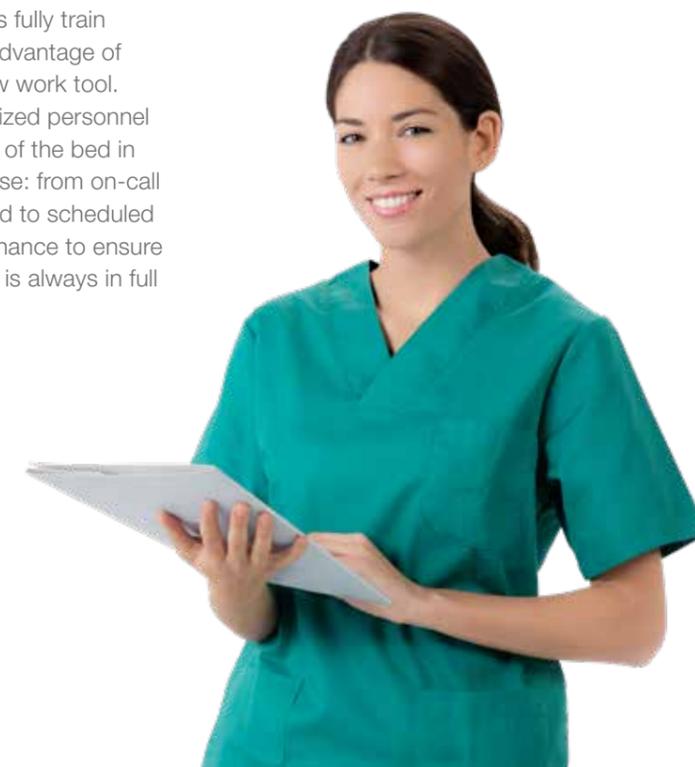The Company reserves the right to introduce all the technical and/or aesthetic modifications deemed necessary to their products without prior notice.

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MALVESTIO S.p.A.